Passive Hand

Product Manual

Fillauer®

Table of Contents

English	3
Spanish.....	8
French	13
German	18
Italian	23
Swedish	28
Norwegian.....	33
Finnish	38
Danish	43
Dutch	48

Intended Use/Purpose

The Fillauer Passive Hand is part of a passive prosthetic arm system consisting of an inner hand with several distal connection options and wire reinforced finger stiffness options. The Passive Hand is designed to be used in combination with our Fillauer endo-arm modular system and our Fillauer cosmetic gloves which are available in varying sizes, structures and colors.

The device is intended for single patient use only.

Performance Characteristics

The Fillauer Passive Hand is constructed of lightweight durable materials and flexible fingers.

Storage and Handling

Do not place this product in direct sunlight for long periods of time. When not in use, store in a cool, clean, dry and dark place.

Warning and Precautions

WARNING: Using acetone or similar chemical solvents will damage our Passive Hands/Cosmetic Gloves and void the warranty.

WARNING: Do not use silicone spray when donning the glove on the Passive Hand.

WARNING: Do not use color pigments when donning the glove on the Passive Hand.

NOTICE: If the Cosmetic Glove has any holes or tears, contact your health care provider.

NOTICE: Abnormal or improper environmental conditions will lead to malfunction and damage of the Passive Hand and is not covered under the warranty of the device. This cosmetic cover must not be subjected to dust/debris, harsh chemicals, abrasives, vibration, activities which would damage the biological limb, or prolonged exposure to extreme temperatures (<-15C or >50C) (<5F or >122F).

NOTICE: The ability of an upper limb prosthetic device user to drive should be determined on a case-by-case basis by a specialist. Contact your local governing authorities regarding any driving restrictions or limitations.

Specifications and Preparations Before Use

Ordering Guide Passive Hands

HAND TYPE		HAND/GLOVE NUMBER	INTERNAL THREAD M12 X 1,5	EXTERNAL THREAD 1/2" -20	EXTERNAL THREAD M12 X 1,5
Passive Hand Standard	5	yyyy	000	010	020
Passive Hand Reinforced	5	yyyy	001	011	021
Passive Hand Extra Reinforced	5	yyyy	003	013	023
Passive Hand Super Reinforced	5	yyyy	002	012	022

*Example: Passive Hand reinforced, internal thread/hand number 30915
= product number 530 915 001*

Consumable Components

Passive Hands are used together with Fillauer cosmetic gloves.

Combination of Devices

The Fillauer Passive Hand can be used with other manufacturers' wrists if the proper stud is used to connect the hand to the wrist. Fillauer Passive Hands are only tested with Fillauer wrists and gloves.

Disposal/waste handling

The product must be disposed of in accordance with applicable local laws and regulations. If the product has been exposed to bacteria or other infectious agents, it must be disposed of in accordance with applicable laws and regulations for the handling of contaminated material.

User Information

The providing health care professional must review the following information directly with the user:

Warnings and Precautions for the user

WARNING: Using acetone or similar chemical solvents will damage our Passive Hands/Cosmetic Gloves and void the warranty.

WARNING: Do not use silicone spray when donning the glove on the Passive Hand.

WARNING: Do not use color pigments when donning the glove on the Passive Hand.

NOTICE: If the Cosmetic Glove has any holes or tears, contact your health care provider.

NOTICE: Abnormal or improper environmental conditions will lead to malfunction and damage of the Passive Hand and is not covered under the warranty of the device. This cosmetic cover must not be subjected to dust/debris, harsh chemicals, abrasives, vibration, activities which would damage the biological limb, or prolonged exposure to extreme temperatures ($<-15^{\circ}\text{C}$ or $>50^{\circ}\text{C}$) ($<5^{\circ}\text{F}$ or $>122^{\circ}\text{F}$).

NOTICE: The ability of an upper limb prosthetic device user to drive should be determined on a case-by-case basis by a specialist. Contact your local governing authorities regarding any driving restrictions or limitations.

Consultation of healthcare professional

The user should monitor their prosthesis daily and contact their health care professional if they experience changes in performance of the device or noise.

Care and Maintenance

No special care is required for the Passive Hand.
Clean the prosthetic glove daily with the use of a mild soap.

Serious Incidents

In the unlikely event a serious incident occurs in relation to the use of the device, users should seek immediate medical help and contact their prosthetist, local competent authority and Fillauer at the earliest possible convenience. Clinicians should at any time contact their local Fillauer representative and local competent authority immediately in the event of any device failure.

Warranty

Warranty is 6 months against manufacturer defects.

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© 2023 Fillauer LLC