Natural Definition Gloves

Product Manual

Fillauer®

Table of Contents

English	3
Spanish.....	8
French	13
German	18
Italian	23
Swedish	28
Norwegian.....	33
Finnish	38
Danish	43
Dutch	48

Intended Use/Purpose

The Natural Definition Glove is a cosmetic glove for upper limb prosthetics. Using a dual layer PVC with added fibers and airbrush technique Natural Definition Glove has a deeper definition and a more natural look. Available in a red and yellow scale with eight colors in each scale, Natural Definition Glove is available in 16 colors covering most skin tones. The Glove is treated with a Micro Coated surface making it more durable and stain resistant compared to other skins.

Natural Definition Glove is intended for single patient use only.

Performance Characteristics

- Airbrushed dual layer PVC with fibers for natural definition
- Skin friendly
- Micro-coated for durability and easy cleaning
- Low fabric friction
- Recyclable - free from all heavy metals and phthalates
- Water resistant

Material

- PVC (free from all heavy metals and phthalates)
- MCV (free from all heavy metals and phthalates)

Natural Definition Guide

For your convenience we made the Natural Defintion Guide to help determine which color to choose for your patient. The guide consists of two different scales, R and Y (Red and Yellow). Each has 8 different colors to choose from to cover most skin tones.

Product number 611200003

Storage and Handling

Do not place this product in direct sunlight for long periods of time. When not in use, store in a cool, clean, dry, and dark place.

Warnings and Precautions

WARNING: Using acetone or similar chemical solvents will damage passive hands/cosmetic gloves and will void the warranty.

WARNING: Do not use silicone spray when donning Natural Definition Glove on the prosthesis.

WARNING: Do not expose the Natural Definition Glove to color pigments when donning Glove on the prosthesis.

WARNING: Be careful not to overheat the skin, max. 100° C. Overheating will damage the vinyl, deform underlying foam, and void the warranty.

NOTICE: Attachment and removal of this cosmetic cover must be performed by or under the direct supervision of a qualified prosthetist. Any adjustment or modifications should be done by the clinician and not by the user.

NOTICE: Abnormal or improper environmental conditions will lead to malfunction and damage of the cosmetic cover and is not covered under the warranty of the device. This cosmetic cover must not be subjected to dust/debris, harsh chemicals, abrasives, vibration, activities which would damage the biological limb, or prolonged exposure to extreme temperatures (<-15C or >50C) (<5F or >122F).

Specifications and Preparations Before Use

Install product on the attached, upper prosthetic. Pull the glove firmly onto the prosthesis. Heat the glove to approximately 60° Celsius or 140°Fahrenheit using hot water or air. The glove will then shrink to its original size. It is not necessary to glue skin in any area.

NOTICE: Use Talcum Powder when donning the glove on the passive hand.

Combination of Devices

See Fillauer website for our full assortment of passive, mechanical and myo hands that work together with our gloves. Fillauer gloves can also be used with other manufacturers' hands.

Disposal/waste handling

The product must be disposed of in accordance with applicable local laws and regulations. If the product has been exposed to bacteria or other infectious agents, it must be disposed of in accordance with applicable laws and regulations for the handling of contaminated material.

If possible, recycle the Natural Definition Glove as plastic.

User Information

The providing health care professional must review the following information directly with the user:

Attachment and removal of this cosmetic cover must be performed by or under the direct supervision of a qualified prosthetist.

Any adjustment or modifications should be done by the clinician and not by the user.

Care and Maintenance

Natural Definition Glove features a unique microcoating, designed to resist staining with normal washing. Clean with soap and water or rubbing alcohol(70%).

WARNING: Do not use Acetone, MEK, or any other solvents..Solvents will damage the coating and void the warranty.

NOTICE: Although our unique micro-coating resists most forms of marks, it is not resistant to all staining agents. For best results, skin should be cleaned as soon as possible after marks are incurred to reduce the possibility of permanent staining. Marks left overnight may become permanent.

NOTICE: If the cosmetic cover has any holes or tears, contact your health care provider.

Serious Incidents

In the unlikely event a serious incident occurs in relation to the use of the device, users should seek immediate medical help and contact their prosthetist, local competent authority and Fillauer at the earliest possible convenience. Clinicians should at any time contact their local Fillauer representative and local competent authority immediately in the event of any device failure.

Warranty

This product has a 6 month warranty against manufacturer defects.

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© 2023 Fillauer LLC