Absolute Sleeve/Liner

Product Manual

Fillauer®

Table of Contents

English	3
Spanish.....	9
French	15
German	21
Italian	27
Swedish	33
Norwegian.....	39
Finnish	45
Danish	51
Dutch	57

Intended Use/Purpose

Absolute Sleeve is intended to be used with a lower limb prosthesis as a suspension sleeve that will make a vacuum within the prosthesis.

Absolute liner is intended to be used with a lower limb prosthesis together with an appropriate suspension.

Absolute Sleeve and Absolute Liner are exceptionally comfortable and durable which combines intimate fit with the latest in sleeve and liner technology. Extra cushioning combines superior comfort with protection for the wearer, while our soft, low friction surface allows it to slide easily against clothing. Medical grade mineral oil is infused in Absolute Sleeve's and Absolute Liner's gel polymer to make sure that skin is cared for with our unique, enriched treatments. The gel is ISO-tested, and FDA approved and extra soft for comfort and improved wearability. Durability is enhanced by using the highest quality of elastomers and materials.

Performance Characteristics

Sleeve:

The Absolute Preflexed- & Straight Sleeve comes in black and beige. The fabric has a high elasticity giving a better fit and a low friction surface allowing it to slide easily against clothing. The gel is less than 3mm thick and Extra Soft which adheres comfortably to the skin with its tacky surface.

The preflexed sleeve is reinforced on the inside of the knee to ensure a longer life.

Liner:

The Absolute Liner comes in black and beige and either as a cushion or locking liner. It is possible to choose between different thicknesses of the liner. Extra cushioning combines superior comfort with protection for the wearer, while our soft, low friction surface allows it to slide easily against clothing. If the user has a very abnormal stump shape, it is possible to order a custom made liner.

Size chart

Absolute Preflexed Suspension Sleeve 3 mm

SIZE	DISTAL CIRCUMFERENCE	PROXIMAL CIRCUMFERENCE*
Small	8.3 - 9.8 in. (21 - 25 cm)	10.2 - 12.2 in. (26 - 31 cm)
Medium	10.2 - 12.6 in. (26 - 32 cm)	12.2 - 15.0 in. (31 - 38 cm)
Medium+	11.8 - 14.2 in. (30 - 36 cm)	15.0 - 17.3 in. (38 - 44 cm)
Large	12.6 - 15.7 in. (32 - 40 cm)	17.3 - 19.7 in. (44 - 50 cm)
Large+	13.4 - 16.5 in. (34 - 42 cm)	19.7 - 23.6 in. (50 - 60 cm)
X-Large	15.7 - 19.3 in. (40 - 49 cm)	23.6 - 27.6 in. (60 - 70 cm)

***NOTE:** The proximal circumference is measured 15 cm above the mid-patella tendon.
The distal circumference is measured 15 cm below the mid-patella tendon.

Absolute Straight Suspension Sleeve 3 mm

SIZE	DISTAL CIRCUMFERENCE	PROXIMAL CIRCUMFERENCE*
X-Small	3.9 - 6.7 in. (10 - 17 cm)	6.3 - 9.4 in. (16 - 24 cm)
Small	5.9 - 8.7 in. (15 - 22 cm)	9.1 - 13.4 in. (23 - 34 cm)
Medium	8.3 - 11.8 in. (21 - 30 cm)	11.4 - 16.9 in. (29 - 43 cm)
Medium+	9.4 - 13.8 in. (24 - 35 cm)	13.0 - 18.5 in. (33 - 47 cm)
Large	11.0 - 16.5 in. (28 - 42 cm)	14.2 - 20.1 in. (36 - 51 cm)
Large+	12.2 - 18.1 in. (31 - 46 cm)	15.4 - 21.7 in. (39 - 55 cm)
X-Large	13.4 - 19.7 in. (34 - 50 cm)	16.5 - 23.6 in. (42 - 60 cm)
XX-Large	14.2 - 21.7 in. (36 - 55 cm)	17.3 - 27.6 in. (44 - 70 cm)

***NOTE:** The proximal circumference is measured 15 cm above the mid-patella tendon. The distal circumference is measured 15 cm below the mid-patella tendon.

Absolut Cusion Liner and Absolut Locking Liner

SIZE	DISTAL CIRCUMFERENCE 5 CM FROM DISTAL END	PROXIMAL CIRCUMFERENCE 40 CM FROM DISTAL END
X-Small	5.5 - 7.9 in. (14 - 20 cm)	9.1 - 13.0 in. (23 - 33 cm)
Small	7.1 - 9.8 in. (18 - 25 cm)	9.8 - 14.2 in. (25 - 36 cm)
Medium	8.7 - 12.2 in. (22 - 31 cm)	11.8 - 16.9 in. (30 - 43 cm)
Medium+	9.1 - 13.0 in. (23 - 33 cm)	14.2 - 20.5 in. (36 - 52 cm)
Large	10.2 - 14.6 in. (26 - 37 cm)	13.4 - 19.3 in. (34 - 49 cm)
Large+	12.6 - 18.1 in. (32 - 46 cm)	16.1 - 23.2 in. (41 - 59 cm)
X-Large	15.7 - 22.8 in. (40 - 58 cm)	18.5 - 26.8 in. (47 - 68 cm)

Storage and Handling

Do not place this product in direct sunlight for long periods of time. When not in use, store in a cool, clean, dry, and dark place.

Warning and Precautions

WARNING: Absolut Liners and Sleeves has been designed and tested for activities of daily living and not for abnormal activities such as extreme sports.

NOTICE: Absolute Sleeve and Absolute Liner is intended for single patient use only.

NOTICE: Attachment, adjustment, alignment, and delivery of this device must be performed by or under the direct supervision of a qualified prosthetist/orthotist. Unless stated in this manual, any such activities should not be attempted by the user and will potentially void the device warranty.

NOTICE: Abnormal or improper environmental conditions will lead to malfunction and damage of the Sleeves/Liners and is not covered under the warranty of the device. This prosthetic device must not be subjected to dust/debris, harsh chemicals, abrasives, vibration, activities which would damage the biological limb, or prolonged exposure to extreme temperatures (<-15C or >50C) (<5F or >122F). Do not allow debris or liquids to remain in the prosthesis/orthosis and its components during use.

NOTICE: If you experience swelling, blisters, or skin irritation, stop wearing the product and contact your health care provider.

Specifications and Preparations Before Use

How to select the right type of Absolute Sleeve or Liner is to look at the shape of the stump and thigh. But also, being aware of prominent bones or other sensitive areas on the stump/leg. Choose the appropriate size to fit the user, and do not forget that it is thermoformable, Absolute Sleeve and Liner can be custom adapted to the precise body shape of each individual patient. For casts with severe concave areas, e.g. "pear shaped" limbs, you might have to vacuum form the liner or order a custom made liner according to your specifications.

Thermoforming Liner

Preparation - Use a dry positive cast without sharp edges. Don a sock on to the cast in order to get an air channel and prevent "airlock" that can damage the gel in the Absolute Liner.

1. Don the Absolute Liner on the room temperature cast
2. Put the oven temperature to 80°C. Avoid bumping the liner against the oven walls as this can damage the gel.
3. After 50 minutes take out the cast with the liner on and leave it to cool down at room temperature.

DO NOT cool down the liner under cold water as this can damage the gel.

Disposal/waste handling

The product must be disposed of in accordance with applicable local laws and regulations. If the product has been exposed to bacteria or other infectious agents, it must be disposed of in accordance with applicable laws and regulations for the handling of contaminated material.

User Information

The providing health care professional must review the following information directly with the user:

Attachment, alignment, and delivery of this orthotic/prosthetic device must be performed by or under the direct supervision of a qualified prosthetist. Any adjustment or modifications should be done by the clinician and not by the user.

Care and Maintenance

Use a mild soap solution or alcohol to clean Absolute Liner/Sleeve. Rinse thoroughly after cleaning to prevent any residue from being retained as this could cause skin irritation.

WARNING: Using acetone or similar chemical solvents will damage Absolute Sleeve/Liner and void the warranty.

Preparations Before Use

Sleeve:

Make sure that the last 5 cm of the sleeve is in contact with the skin of the thigh to be able to create the vacuum that will give you a better suspension.

Liner:

Make sure that there is full contact and no air in the bottom between the skin and the liner.

Consultation of Health Care Professional

The user should monitor their prosthesis daily and contact their health care professional if they experience changes in performance of the device or hear irregular noise from it.

The user should monitor skin condition daily and contact their health care professional immediately if the device causes skin damage or irritation.

Serious Incidents

In the unlikely event a serious incident occurs in relation to the use of the device, users should seek immediate medical help and contact their prosthetist, local competent authority and Fillauer at the earliest possible convenience. Clinicians should at any time contact their local Fillauer representative and local competent authority immediately in the event of any device failure.

Liability

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the Instructions for Use. The device must only be operated with tested modular components. The manufacturer is not liable for damage caused by component combinations that were not authorised by the manufacturer.

Warranty

Sleeves and Custom Liners/ Sleeves have a 3-month warranty against manufacturer defects.

Linners have a 6-month warranty against manufacturer defects.

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© 2023 Fillauer LLC