


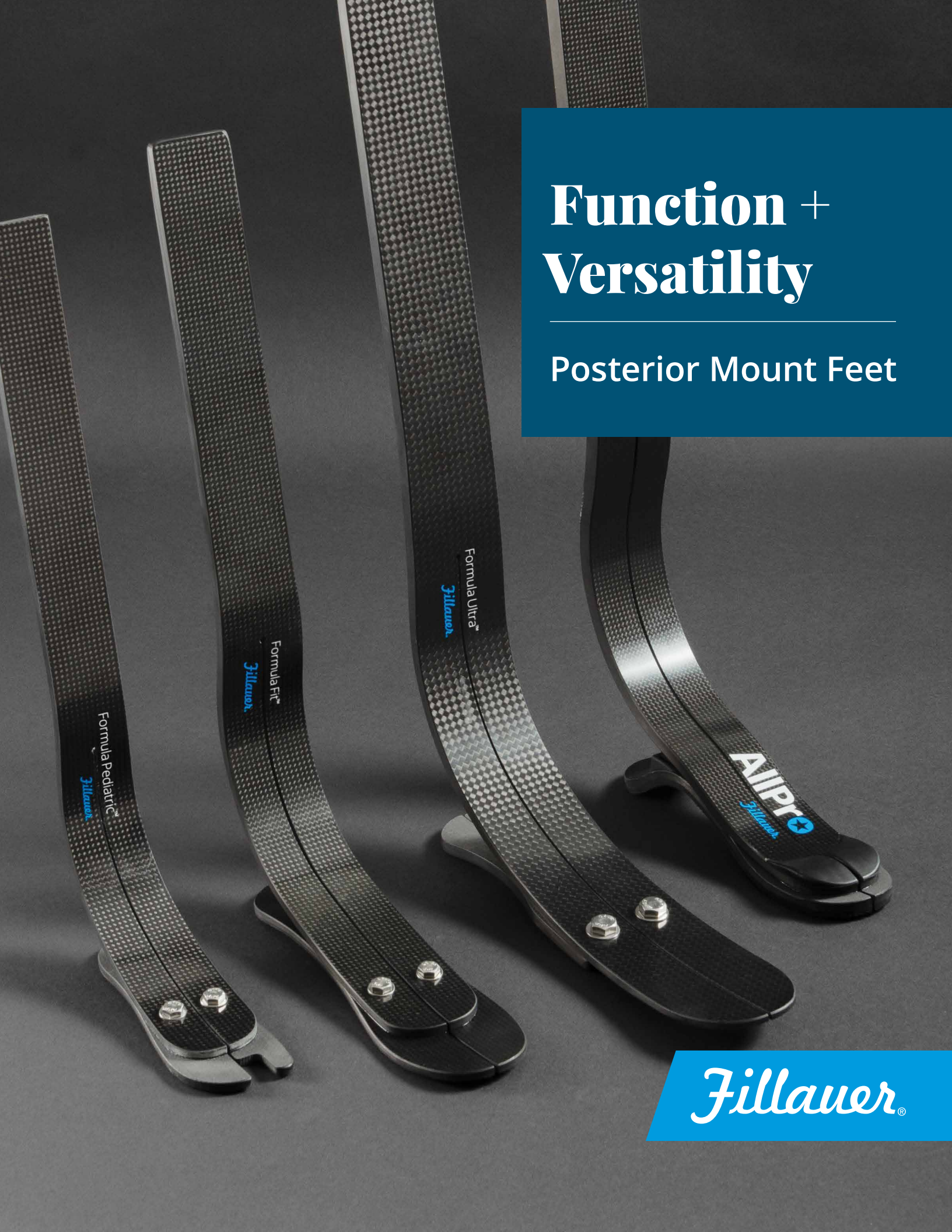

# Function + Versatility

Posterior Mount Feet

*Jillauer*®


## Formula Pediatric™

### Get ready to grow.


A first of its kind, the **Formula Pediatric** is a high-performance, customizable, pediatric foot that grows with the child. The long, lightweight carbon pylon provides critical energy return through the posterior attachment, while the compact shape of the ankle simplifies cosmetic finishing without hindering performance.



Split Pylon with Sandal Toe

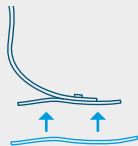

Replaceable Foot Plate for Growth

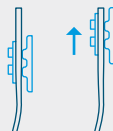

Optional Custom Pylon for Vertical Growth




## Formula Fit™

### The perfect balance.


Bridging the gap between the Formula Ultra and Formula Pediatric, the **Formula Fit** is the perfect solution for adolescents and small adults who want to be active. Designed to be used with Fillauer's standard foot shell, the foot accommodates K3 to high K4 activity users.



Split Pylon fits Standard Fillauer Foot Shell

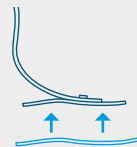

Replaceable Foot Plate for Growth

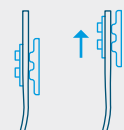

Optional Custom Pylon for Vertical Growth

# Formula Pediatric & Formula Fit

|                   | Formula Pediatric                       | Formula Fit              |
|-------------------|-----------------------------------------|--------------------------|
| Patient Weight    | 125 lbs. (57 kg)                        | 176 lbs. (80 kg)         |
| Foot Build Height | 1.5 in. (3.8 cm)                        | 2 in. (5.1 cm)           |
| Size              | 16 – 24 cm                              | 24 – 28 cm               |
| Product Weight    | 11.6 oz. (330 g)                        | 15.9 oz. (452 g)         |
| Functional Level  | K3 – K4                                 | K3 – K4                  |
| Primary Materials | Carbon Composite                        | Carbon Composite         |
| Sandal Toe        | Yes                                     | No                       |
| Foot Shell        | College Park / Össur Foot Shells        | Fillauer Foot Shell      |
| Durable           | Meets ISO-22675 Standard                | Meets ISO-22675 Standard |
| Waterproof        | The foot unit is waterproof to 1 meter. |                          |
| Warranty          | 24 months                               | 24 months                |

## Formula Pediatric

College Park / Össur foot shell sold separately.

- 178-120-XX-XXL** Formula Pediatric Left  
**178-120-XX-XXR** Formula Pediatric Right  
**178-120-XXC-XXL** Formula Pediatric Left with Custom Pylon  
**178-120-XXC-XXR** Formula Pediatric Right with Custom Pylon



| Patient Weight |            | 16 cm           | 17 cm           | 18 cm           | 19 cm           | 20 cm           | 21 cm           | 22 cm           | 23 cm           | 24 cm           |
|----------------|------------|-----------------|-----------------|-----------------|-----------------|-----------------|-----------------|-----------------|-----------------|-----------------|
| 30 – 45 lbs.   | 14 – 20 kg | 5A-16<br>5AC-16 | 5A-17<br>5AC-17 | 5A-18<br>5AC-18 | 5A-19<br>5AC-19 | 5A-20<br>5AC-20 | 5A-21<br>5AC-21 | 5A-22<br>5AC-22 | 5A-23<br>5AC-23 | 5A-24<br>5AC-24 |
| 46 – 60 lbs.   | 21 – 27 kg | 4A-16<br>4AC-16 | 4A-17<br>4AC-17 | 4A-18<br>4AC-18 | 4A-19<br>4AC-19 | 4A-20<br>4AC-20 | 4A-21<br>4AC-21 | 4A-22<br>4AC-22 | 4A-23<br>4AC-23 | 4A-24<br>4AC-24 |
| 61 – 75 lbs.   | 28 – 34 kg | 3A-16<br>3AC-16 | 3A-17<br>3AC-17 | 3A-18<br>3AC-18 | 3A-19<br>3AC-19 | 3A-20<br>3AC-20 | 3A-21<br>3AC-21 | 3A-22<br>3AC-22 | 3A-23<br>3AC-23 | 3A-24<br>3AC-24 |
| 76 – 99 lbs.   | 35 – 45 kg | 2A-16<br>2AC-16 | 2A-17<br>2AC-17 | 2A-18<br>2AC-18 | 2A-19<br>2AC-19 | 2A-20<br>2AC-20 | 2A-21<br>2AC-21 | 2A-22<br>2AC-22 | 2A-23<br>2AC-23 | 2A-24<br>2AC-24 |
| 100 – 125 lbs. | 45 – 57 kg | A-16<br>AC-16   | A-17<br>AC-17   | A-18<br>AC-18   | A-19<br>AC-19   | A-20<br>AC-20   | A-21<br>AC-21   | A-22<br>AC-22   | A-23<br>AC-23   | A-24<br>AC-24   |

## Formula Fit

Foot Shell sold separately.

- 178-130-X-XX** Formula Fit  
**178-130-XC-XX** Formula Fit with Custom Pylon

| Patient Weight |            | 24 cm         | 25 cm         | 26 cm         | 27 cm         | 28 cm         |
|----------------|------------|---------------|---------------|---------------|---------------|---------------|
| 70 – 95 lbs.   | 32 – 43 kg | A-24<br>AC-24 | A-25<br>AC-25 | A-26<br>AC-26 | A-27<br>AC-27 | A-28<br>AC-28 |
| 95 – 120 lbs.  | 43 – 54 kg | B-24<br>BC-24 | B-25<br>BC-25 | B-26<br>BC-26 | B-27<br>BC-27 | B-28<br>BC-28 |
| 120 – 145 lbs. | 54 – 66 kg | C-24<br>CC-24 | C-25<br>CC-25 | C-26<br>CC-26 | C-27<br>CC-27 | C-28<br>CC-28 |
| 145 – 176 lbs. | 66 – 80 kg | D-24<br>DC-24 | D-25<br>DC-25 | D-26<br>DC-26 | D-27<br>DC-27 | D-28<br>DC-28 |



## Formula Pediatric and Formula Fit Accessories

| Product Number     | Description                  |
|--------------------|------------------------------|
| <b>180-10-5000</b> | Adjustable Alignment Bracket |
| <b>180-10-3000</b> | Posterior Mounting Bracket   |
| <b>180-10-5010</b> | AAB Wedge Kit                |





## AllPro PM

AllPro performance with only 4 inches of build height.

The **AllPro PM** redefines versatility, blending exceptional flexibility with dynamic performance to create a prosthetic foot that empowers users to work and play like a pro. Designed for seamless transitions, this all-purpose foot effortlessly moves from walking to competing, adapting to the demands of the gym, basketball court, soccer field, and beyond.



Designed for active to competitive adults



Compatible with Fillauer Foot Shell or ProCover

## Formula Ultra™

Ultra custom, ultra performance.

The **Formula Ultra** is the ideal balance of flexibility and power, creating a dynamic foot design intended for a wide range of users. The long, lightweight carbon pylon provides critical energy return to the user through the posterior attachment, while the compact shape of the ankle allows for better cosmetic finishing without hindering foot performance while worn with a shoe.

Designed for competitive adults



Foot cover custom fabricated by prosthetist



# AllPro PM & Formula Ultra

|                   | AllPro PM                               | Formula Ultra            |
|-------------------|-----------------------------------------|--------------------------|
| Patient Weight    | 330 lbs. (150 kg)                       | 330 lbs. (150 kg)        |
| Foot Build Height | 4 in. (10.2 cm)                         | 3 in. (7.6 cm)           |
| Size              | 22 – 30 cm                              | 26 – 31 cm               |
| Product Weight    | 21 oz. (600 g)                          | 24 oz. (675 g)           |
| Functional Level  | K3 – K4                                 | K3 – K4                  |
| Primary Materials | Carbon Composite                        | Carbon Composite         |
| Sandal Toe        | No                                      | No                       |
| Foot Shell        | Fillauer Foot Shell / ProCover          | Custom                   |
| Durable           | Meets ISO-22675 Standard                | Meets ISO-22675 Standard |
| Waterproof        | The foot unit is waterproof to 1 meter. |                          |
| Warranty          | 36 months                               | 24 months                |



## AllPro PM

Foot Shell sold separately.

**137-PM-XX-XX** AllPro Foot, Posterior Mount

| Patient Weight |              | 22 cm | 23 cm | 24 cm | 25 cm | 26 cm | 27 cm | 28 cm | 29 cm  | 30 cm  |
|----------------|--------------|-------|-------|-------|-------|-------|-------|-------|--------|--------|
| 100 – 119 lbs. | 45 – 54 kg   | 22-A2 | 23-A2 | 24-A2 | 25-B4 | 26-B4 |       |       |        |        |
| 120 – 139 lbs. | 54 – 63 kg   | 22-A3 | 23-A3 | 24-A3 | 25-B4 | 26-B4 | 27-C6 | 28-C6 |        |        |
| 140 – 159 lbs. | 64 – 72 kg   | 22-B4 | 23-B4 | 24-B4 | 25-B4 | 26-B4 | 27-C6 | 28-C6 | 29-D7  | 30-D7  |
| 160 – 179 lbs. | 73 – 81 kg   | 22-B4 | 23-B4 | 24-B4 | 25-C5 | 26-C5 | 27-C6 | 28-C6 | 29-D7  | 30-D7  |
| 180 – 199 lbs. | 82 – 90 kg   | 22-C5 | 23-C5 | 24-C5 | 25-C5 | 26-C5 | 27-D7 | 28-D7 | 29-D7  | 30-D7  |
| 200 – 224 lbs. | 91 – 102 kg  | 22-C5 | 23-C5 | 24-C5 | 25-D6 | 26-D6 | 27-D7 | 28-D7 | 29-E8  | 30-E8  |
| 225 – 249 lbs. | 102 – 113 kg |       |       |       | 25-D6 | 26-D6 | 27-E8 | 28-E8 | 29-E8  | 30-E8  |
| 250 – 275 lbs. | 113 – 125 kg |       |       |       | 25-E7 | 26-E7 | 27-E8 | 28-E8 | 29-F9  | 30-F9  |
| 276 – 300 lbs. | 125 – 136 kg |       |       |       | 25-E7 | 26-E7 | 27-F9 | 28-F9 | 29-F9  | 30-F9  |
| 301 – 330 lbs. | 137 – 150 kg |       |       |       | 25-F9 | 26-F9 | 27-F9 | 28-F9 | 29-G10 | 30-G10 |

## Formula Ultra

**178-110-XX-XX** Formula Ultra

| Patient Weight |              | Soft         | Standard     | Stiff        |
|----------------|--------------|--------------|--------------|--------------|
| 100 – 129 lbs. | 45 – 58 kg   |              | 178-110-2A-5 | 178-110-2A-6 |
| 130 – 159 lbs. | 59 – 72 kg   | 178-110-A-5  | 178-110-A-6  | 178-110-A-7  |
| 160 – 189 lbs. | 73 – 85 kg   | 178-110-B-6  | 178-110-B-7  | 178-110-B-8  |
| 190 – 219 lbs. | 86 – 99 kg   | 178-110-C-7  | 178-110-C-8  | 178-110-C-9  |
| 220 – 249 lbs. | 100 – 113 kg | 178-110-D-8  | 178-110-D-9  | 178-110-D-10 |
| 250 – 289 lbs. | 114 – 131 kg | 178-110-E-9  | 178-110-E-10 | 178-110-E-11 |
| 290 – 330 lbs. | 132 – 150 kg | 178-110-F-10 | 178-110-F-11 |              |



## AllPro PM and Formula Ultra Accessories

| Product Number     | Description                  |
|--------------------|------------------------------|
| <b>180-10-4000</b> | Adjustable Alignment Bracket |
| <b>180-10-2010</b> | Posterior Mounting Bracket   |
| <b>180-10-4010</b> | AAB Wedge Kit                |



## Obsidian


|                           |                                                              |
|---------------------------|--------------------------------------------------------------|
| Obsidian Adult Running DM |                                                              |
| <b>K4</b>                 | <b>Build Height</b><br>10.75 in.<br>27.3 cm                  |
| <b>K3</b>                 | <b>Patient Weight</b><br>330 lbs.<br>150 kg                  |
|                           | <b>Recommended L-Codes*</b><br>L5999<br><b>PDAC Approved</b> |
|                           | <b>Product Weight</b><br>19.4 oz.<br>550 g                   |



|                               |                                               |
|-------------------------------|-----------------------------------------------|
| Obsidian Pediatric Running DM |                                               |
| <b>K4</b>                     | <b>Build Height</b><br>6.3 in.<br>160 mm      |
| <b>K3</b>                     | <b>Patient Weight</b><br>124 lbs.<br>56 kg    |
|                               | <b>Recommended L-Codes*</b><br>L5980<br>L5999 |
|                               | <b>Product Weight</b><br>8.5 oz.<br>240 g     |



|                           |                                                              |
|---------------------------|--------------------------------------------------------------|
| Obsidian Adult Running PM |                                                              |
| <b>K4</b>                 | <b>Build Height</b><br>2.0 in.<br>5.1 cm                     |
| <b>K3</b>                 | <b>Patient Weight</b><br>330 lbs.<br>150 kg                  |
|                           | <b>Recommended L-Codes*</b><br>L5999<br><b>PDAC Approved</b> |
|                           | <b>Product Weight</b><br>28 oz.<br>800 g                     |

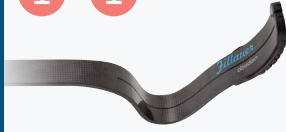

|                               |                                               |
|-------------------------------|-----------------------------------------------|
| Obsidian Pediatric Running PM |                                               |
| <b>K4</b>                     | <b>Build Height</b><br>2 in.<br>5.1 cm        |
| <b>K3</b>                     | <b>Patient Weight</b><br>180 lbs.<br>82 kg    |
|                               | <b>Recommended L-Codes*</b><br>L5980<br>L5999 |
|                               | <b>Product Weight</b><br>19.4 oz.<br>550 g    |



## AIIPro

|            |                                                                       |
|------------|-----------------------------------------------------------------------|
| AIIPro XTS |                                                                       |
| <b>K4</b>  | <b>Build Height</b><br>6.4 (16.2 cm)<br>8.4 in. (21.3 cm)             |
| <b>K3</b>  | <b>Patient Weight</b><br>276 lbs. (125 kg)                            |
|            | <b>Sizes</b><br>22 – 30 cm                                            |
|            | <b>Recommended L-Codes*</b><br>L5987<br>L5984<br><b>PDAC Approved</b> |
|            | <b>Product Weight</b><br>25.6 oz. (728 g)<br>27.8 oz. (789 g)         |



|           |                                                              |
|-----------|--------------------------------------------------------------|
| AIIPro DM |                                                              |
| <b>K4</b> | <b>Build Height</b><br>6 in. (15.2 cm)<br>8 in. (20.3 cm)    |
| <b>K3</b> | <b>Patient Weight</b><br>330 lbs. (150 kg)                   |
|           | <b>Sizes</b><br>22 – 30 cm                                   |
|           | <b>Recommended L-Codes*</b><br>L5981<br><b>PDAC Approved</b> |
|           | <b>Product Weight</b><br>19.4 oz.<br>550 g                   |



|                     |                                                           |
|---------------------|-----------------------------------------------------------|
| AIIPro DM Pediatric |                                                           |
| <b>K4</b>           | <b>Build Height</b><br>6 in. (15.2 cm)<br>8 in. (20.3 cm) |
| <b>K3</b>           | <b>Patient Weight</b><br>119 lbs. (54 kg)                 |
|                     | <b>Sizes</b><br>19 – 21 cm                                |
|                     | <b>Recommended L-Codes*</b><br>L5981                      |
|                     | <b>Product Weight</b><br>9.6 oz.<br>273 g                 |



|           |                                                              |
|-----------|--------------------------------------------------------------|
| AIIPro PM |                                                              |
| <b>K4</b> | <b>Build Height</b><br>4.0 in.<br>10 cm                      |
| <b>K3</b> | <b>Patient Weight</b><br>330 lbs. (150 kg)                   |
|           | <b>Sizes</b><br>22 – 30 cm                                   |
|           | <b>Recommended L-Codes*</b><br>L5980<br><b>PDAC Approved</b> |
|           | <b>Product Weight</b><br>28 oz.<br>800 g                     |



## Formula

|                |                                                              |
|----------------|--------------------------------------------------------------|
| Formula Ultra™ |                                                              |
| <b>K4</b>      | <b>Build Height</b><br>3 in.<br>7.6 cm                       |
| <b>K3</b>      | <b>Patient Weight</b><br>330 lbs. (150 kg)                   |
|                | <b>Sizes</b><br>26 – 31 cm                                   |
|                | <b>Recommended L-Codes*</b><br>L5980<br><b>PDAC Approved</b> |
|                | <b>Product Weight</b><br>24 oz.<br>675 g                     |



|              |                                               |
|--------------|-----------------------------------------------|
| Formula Fit™ |                                               |
| <b>K4</b>    | <b>Build Height</b><br>2 in.<br>5.1 cm        |
| <b>K3</b>    | <b>Patient Weight</b><br>176 lbs. (80 kg)     |
|              | <b>Sizes</b><br>24 – 28 cm                    |
|              | <b>Recommended L-Codes*</b><br>L5980<br>L5986 |
|              | <b>Product Weight</b><br>11.6 oz.<br>330 g    |



|                    |                                               |
|--------------------|-----------------------------------------------|
| Formula Pediatric™ |                                               |
| <b>K4</b>          | <b>Build Height</b><br>1.5 in.<br>3.8 cm      |
| <b>K3</b>          | <b>Patient Weight</b><br>125 lbs. (57 kg)     |
|                    | <b>Sizes</b><br>16 – 24 cm                    |
|                    | <b>Recommended L-Codes*</b><br>L5980<br>L5986 |
|                    | <b>Product Weight</b><br>11.6 oz.<br>330 g    |



|                          |                                                                                    |
|--------------------------|------------------------------------------------------------------------------------|
| Formula Pediatric Custom |                                                                                    |
| <b>K4</b>                | <b>Build Height</b><br>1.5 in.<br>3.8 cm                                           |
| <b>K3</b>                | <b>Patient Weight</b><br>125 lbs. (57 kg)                                          |
|                          | <b>Sizes</b><br>16 – 24 cm                                                         |
|                          | <b>Recommended L-Codes*</b><br>L5980<br>L5986<br>L5910                             |
|                          | <b>Product Weight</b><br>15 oz. (425 g)<br><b>Weight includes mounting bracket</b> |



**K4**



**Build Height**  
3.0 in.  
7.6 cm

**Patient Weight**  
275 lbs.  
125 kg

**Product Weight**  
35 oz.  
1000 g

**Recommended L-Codes\***  
L5999

## Aeris Performance LP

**K3**



**Build Height**  
5.1 in.  
13 cm

**Patient Weight**  
276 lbs. (125 kg)

**Sizes**  
22 – 30 cm

**Product Weight**  
15.9 oz.  
450 g

**Recommended L-Codes\***  
L5981  
L5986

## Wave Comfort

**K3**



**Build Height**  
4 in.  
10 cm

**Patient Weight**  
330 lbs (150 kg)

**Sizes**  
22 – 30 cm

**Product Weight**  
13.2 oz.  
375 g

**Recommended L-Codes\***  
L5981  
L5986

## Aeris Performance 2

**K4**



**Build Height**  
6.5 in.  
16.5 cm

**Patient Weight**  
330 lbs (150 kg)

**Sizes**  
22 – 30 cm

**Product Weight**  
18.5 oz.  
525 g

**Recommended L-Codes\***  
L5981  
**PDAC Approved**

## Motion Foot

**K3**



**Build Height**  
4.1 in.  
10.4 cm

**Patient Weight**  
276 lbs. (125 kg)

**Sizes**  
22 – 30 cm

**Product Weight**  
22.7 oz.  
644 g

**Recommended L-Codes\***  
L5981  
L5968

## Ibex

**K4**



**Build Height**  
7.2 in.  
18.4 cm

**Patient Weight**  
440 lbs (200 kg)

**Sizes**  
25 – 30 cm

**Product Weight**  
21.2 oz.  
602 g

**Recommended L-Codes\***  
L5980  
L5986

\*Suggested L-Codes are provided as a reference only. It is the responsibility of the practitioner and/or patient care facility submitting this claim to determine and submit the correct coding within all established rules and guidelines.

# Cosmetic Cover Options

## ProCover — AllPro PM Only



### Split Design

ProCover is easy to install and remove for cleaning and maintenance.



### Innovative Materials

The durable polyurethane foam maintains superior energy return over time — ideal for high activity patients.



### Design-to-Fit Shape

The unique shape enhances rollover performance and provides a secure fit for a more responsive feel.

### Specifications

| Product Weight (26 cm) | Functional Level | Material          | Waterproof            | Warranty |
|------------------------|------------------|-------------------|-----------------------|----------|
| 9.1 oz. (259 g)        | K3 – K4          | Polyurethane foam | Waterproof to 1 meter | 9 months |

### Ordering Information

ProCover purchase includes two hardware kits.

|          | 22 cm              | 23 cm              | 24 cm              | 25 cm              | 26 cm              | 27 cm              | 28 cm              | 29 cm              | 30 cm              |
|----------|--------------------|--------------------|--------------------|--------------------|--------------------|--------------------|--------------------|--------------------|--------------------|
| ProCover | <b>182-10-1022</b> | <b>182-10-1023</b> | <b>182-10-1024</b> | <b>182-10-1025</b> | <b>182-10-1026</b> | <b>182-10-1027</b> | <b>182-10-1028</b> | <b>182-10-1029</b> | <b>182-10-1030</b> |

### Replacement Parts

|                          | Quantity                           | Size       | Part Number        |
|--------------------------|------------------------------------|------------|--------------------|
| Hardware Replacement Kit | 4 Connector Pins, 2 Velcro® Straps | 22 – 25 cm | <b>182-10-2300</b> |
| Hardware Replacement Kit | 4 Connector Pins, 2 Velcro® Straps | 26 – 30 cm | <b>182-10-2350</b> |

Patent US10874529B2.

## Foot Shell — Formula Fit & AllPro PM Only

Made with BIOVYN™, a bio-attributed PVC that is based on 100% renewable feedstock, our product not only contributes to significantly reduced carbon emissions but also offers enhanced durability. To complement the foot shell, a sleek, black prosthetic sock, sized appropriately, is now included.



CC 03 CC 09 CC 13 CC 99

### Specifications

| Product Weight (26 cm) | Material    | Warranty |
|------------------------|-------------|----------|
| 8.9 oz. (251 g)        | BIOVYN™ PVC | 9 months |

### Ordering Information

Replace **XX** with foot length (22 – 30 cm). Foot shell includes black prosthetic sock.

| Color | Left                   | Right                  |
|-------|------------------------|------------------------|
| 03    | 45- <b>XX</b> -13-03-3 | 46- <b>XX</b> -13-03-3 |
| 09    | 45- <b>XX</b> -13-09-3 | 46- <b>XX</b> -13-09-3 |
| 13    | 45- <b>XX</b> -13-13-3 | 46- <b>XX</b> -13-13-3 |
| 99    | 45- <b>XX</b> -13-99-3 | 46- <b>XX</b> -13-99-3 |

### Prosthetic Socks

| Size       | Part Number      |
|------------|------------------|
| 22 – 24 cm | <b>636202224</b> |
| 25 – 27 cm | <b>636202527</b> |
| 28 – 30 cm | <b>636202830</b> |

*Fillauer*®

### Fillauer LLC

2710 Amnicola Highway  
Chattanooga, TN 37406  
423.624.0946

### Fillauer Europe

Kung Hans Väg 2  
192 68 Sollentuna, Sweden  
+46 8 505 332 00

© 2025 Fillauer LLC  
AD380/06-02-22/05-19-25