



Elbow Assembly


- A. 50720 Short Sleeve
- B. 50721 Long Sleeve
- C. 57497 Internal Assembly, E-400 and HD
- D. 50725 Cable Housing Assembly
- E. 50667 Lock Nut
- F. 50668 Belleville Washer
- G. 50658 Turntable, E-400, 400HD
50659 Turntable, E-400A
- H. 50665 Cork Washer
- I. 50719 Shaft
- J. 50739 Outside Washer
- K. 50740 Screw
- L. 50724 Cable Assembly
- M. 50729 Washer
- N. 62235 Heavy Duty Gear Sector, E-400 and 400A
62236 Gear Sector, E-400HD
62237 Extra Heavy Duty Gear Sector, E-400XD, Powerbow
- O. 50748 Yoke Assembly, E-400 and E-400HD
60143 Yoke Assembly, E-400 XHD
60664 Yoke Assembly, Power Bow
- P. 50730 Elbow Saddle Assembly, E-400 and 400A
50738 Elbow Saddle Assembly, E-400HD
- Q. 50742 Sheet Metal Screw
- R. Elbow Cap—See Table
- S. 50427 Anchor
- T. 50747 Hanger
- U. 50680 Bolt
- V. 50678 Screw
- W. 50722 Bearing

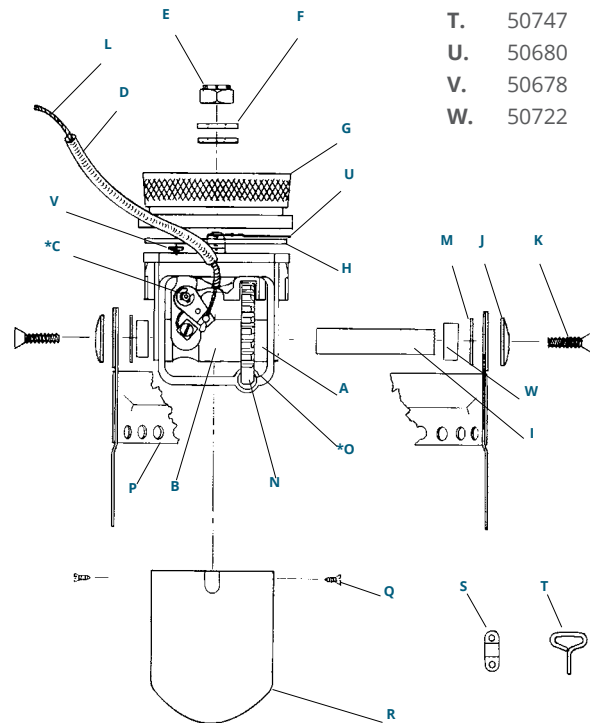


- C. **62243 Internal Assembly, E-400 and E-400HD**
- X. 62242 Internal Cage Assembly, E-400 and E-400HD
- Y. 50677 Base
- Z. 50679 Screw
- AA. 50680 Bolt
- AB. 50715 Locking Bar Spring

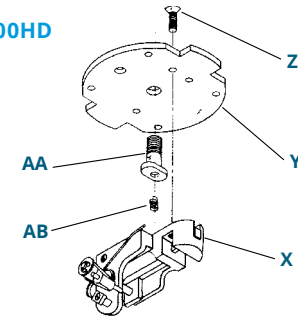

- O. **50748 Yoke Assembly**
- AC. 50671 Yoke
- AD. 50675 Nut
- AE. 50674 Bumper
- AF. 50758 Yoke Cover, Right
50773 Yoke Cover, Left
- AG. 50761 Screw

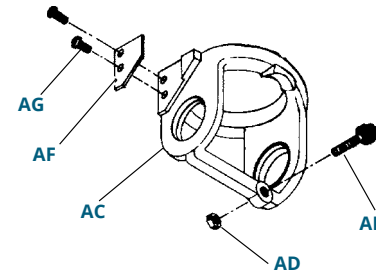


- X. **62242 Internal Cage Assembly, E-400 and E-400HD**
- AH. 62239 Cage
- AI. 50684 Cam Bearing
- AJ. 880108 Screw
- AK. 50687 Cam
- AL. 50689 Spring
- AM. 50701 Washer
- AN. 50160 Retaining Clip
- AO. 50710 Ratchet Assembly
- AP. 50714 Locking Bar
- AQ. 50813 Case Pin
- AR. 50713 Cable Pin
- AS. 55561 Lever Sub Assembly

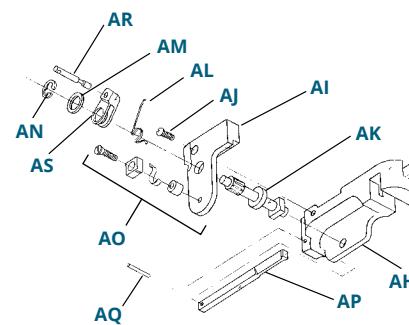


60148 Internal Cage Assembly, E-400XHD and Power Bow (SST Double Gear Sector)

- AO. **50710 Ratchet Assembly**
- AT. 55561 Lever Sub Assembly
- AU. 50711 Spacer
- AV. 50712 Screw
- AW. 50708 Keeper
- AX. 50710 Ratchet

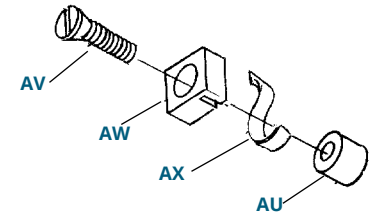

- AT. **55561 Lever Sub Assembly**
- AY. 50630 Pawl
- AZ. 50705 Spring
- BA. 50706 Pin
- BB. 50688 Lever

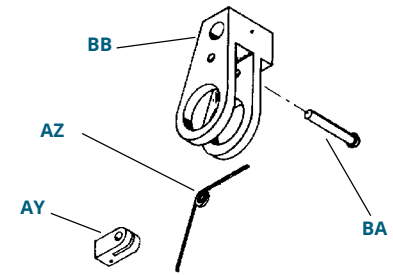

Elbow Cap

Color	Outside Cable	Inside Cable	Friction Elbow
Tan	50645	54437	54436
Light Brown	54435	54405	54433
Medium Brown	54432	54431	54430
Dark Brown	54429	54428	54427
Jet Black	54434	54406	54407