



Instructions for Lever Release Kit

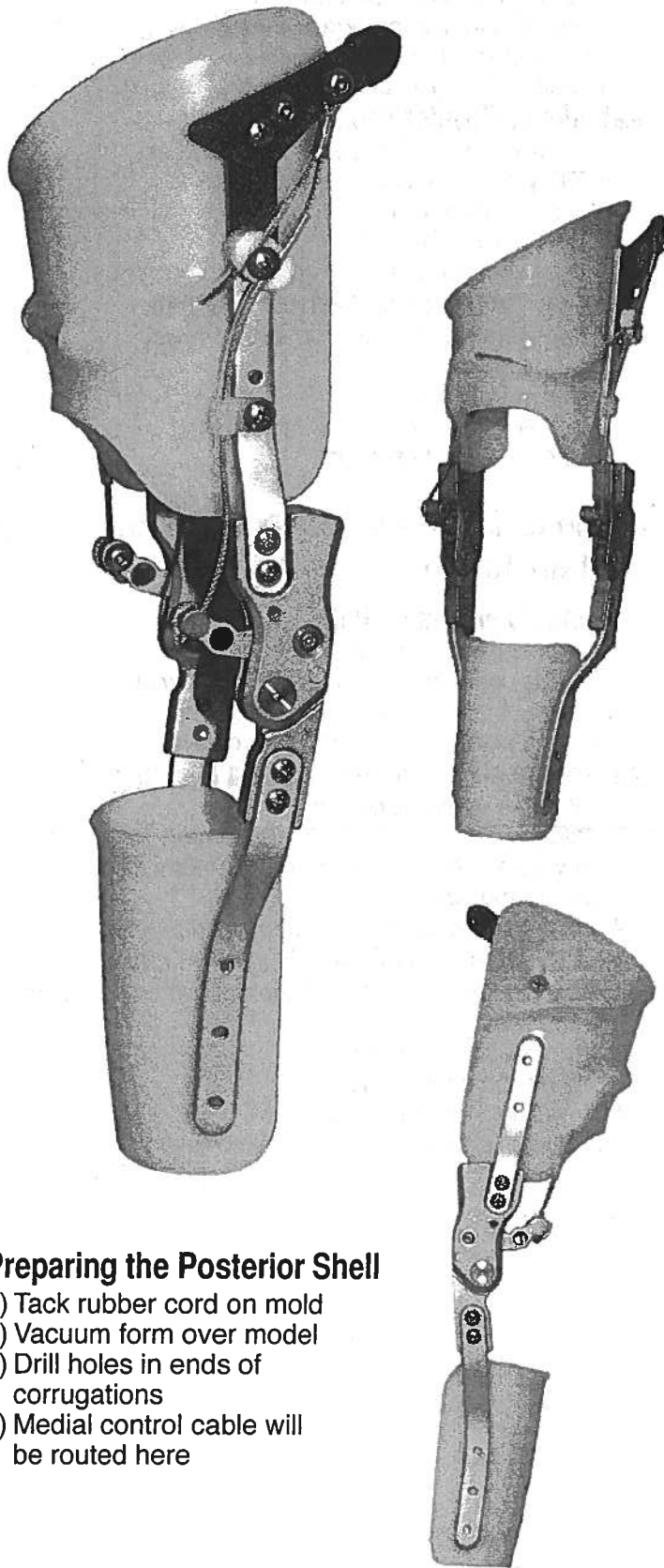


Preparing the Posterior Shell

- 1) Tack rubber cord on mold
- 2) Vacuum form over model
- 3) Drill holes in ends of corrugations
- 4) Medial control cable will be routed here

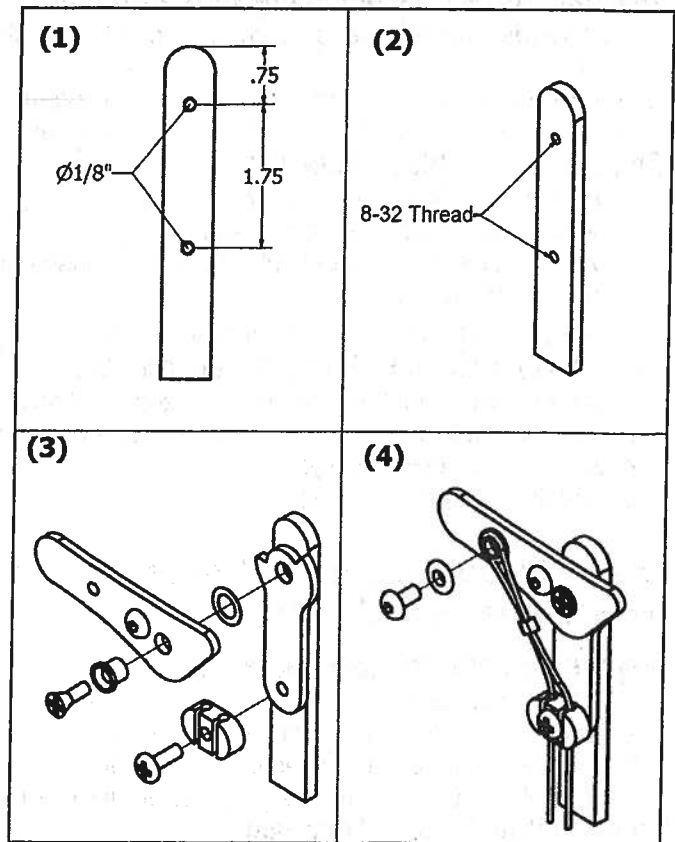

Cable and Handle Attachment Procedure

- 1) Drill lateral side bar with 1/8" drill as shown
- 2) Tap for 8-32 threads
- 3) Assemble as shown
- 4) Route medial cable through corrugations
- 5) Secure cable with hot glue

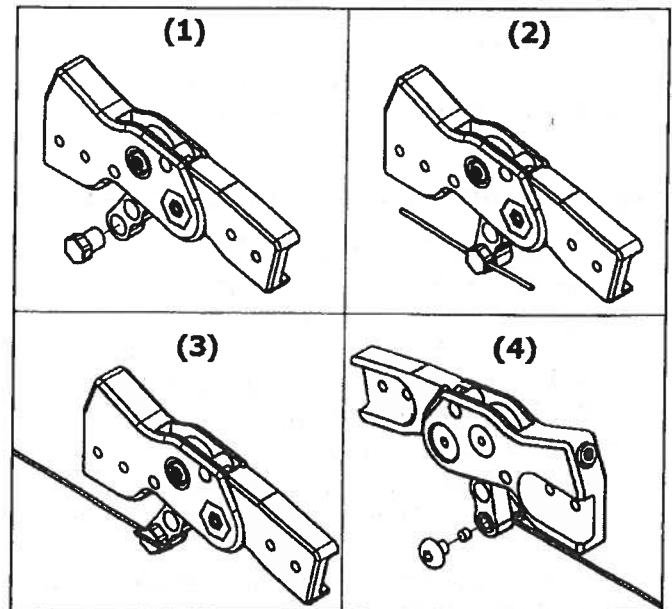

Cable Attachment Procedure

- 1) Insert Pivoting Connector into handle
- 2) Insert cable into cross hole in connector
- 3) Wrap cable around connector and re-insert end through same hole
- 4) Insert rubber plug and tighten screw

Fillauer®
North Carolina

For more detailed instructions: 800-221-4769
info@ots-corp.com • www.fillauer.com