


# Installing the Multi-Flex Wrist Cloth Cover

## Installation Instructions

Please go over the following steps for Installing the Multi-Flex Wrist Cloth Cover.

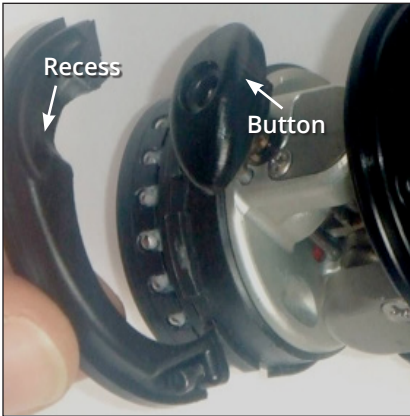

1. Place one of the Half-Rings on the Quick Disconnect and align the recess with one of the oval-shaped buttons.



2. Install the second Half-Ring from the opposite side.



3. Make sure the recesses in the Half-Rings are aligned under the oval buttons. Start tightening both screws using the hex wrench. Check the alignment and make sure top of the Half-Rings is flush with the top of the Quick Disconnect.



4. Orient the oval shape of the Cloth Cover with the oval shape of the installed Half-Rings.




5. Start the Cloth Cover onto the device at an angle. Guide the top ring over the Quick Disconnect and then over the installed Half-Rings.



6. Slide the Cloth Cover up until the bottom ring is against the Half-Rings.



7. Keep the ovals of the Cloth Cover and the device aligned. Stretch the bottom ring up onto the Half-Rings, making sure the flange is in the groove of the bottom ring all the way around the Cover.

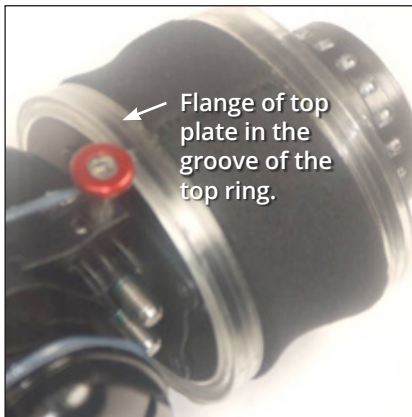

8. Pull the top ring of the Cloth Cover up and onto the top plate of the Multi-Flex Wrist Assembly. Make sure the flange of the top plate is in the groove of the top ring all the way around the Cover.

## Customer Support

If you have any questions or concerns please contact us at Motion Control, a Division of Fillauer: [motioninfo@fillauer.com](mailto:motioninfo@fillauer.com) or 801-326-3434.

## Single Patient Use

Each amputee is unique. The shape of their residual limb, the control signals each generates and the tasks an amputee performs during the day require specialized design and adjustment of the prosthesis. Motion Control products are manufactured to be fit to one individual.

## Risk Management

To minimize the risk of device damage or injury to the user while maximizing the functions of this device, follow the instructions for installation, and use this device as described in this manual. Use caution when using this device in situations where injury to yourself or others may occur. In the unlikely event a serious incident occurs in relation to the use of the device, users should seek immediate medical help.

## Disposal/Waste Handling

This device, including any associated electronics and batteries should be disposed of in accordance with applicable local laws and regulations. This includes laws and regulations regarding bacterial or infectious agents, if necessary.

## Declaration of Conformity

The product herewith complies with Medical Device Regulation 2017/745 and is registered with the United States Food and Drug Administration. (Registration No. 1723997)



## Customer Support

### Americas, Oceania, Japan

#### Fillauer Motion Control


115 N. Wright Brothers Drive  
Salt Lake City, UT 84116  
801-326-3434  
[motioninfo@fillauer.com](mailto:motioninfo@fillauer.com)

### Europe, Africa, Asia

#### Fillauer Europe

Kung Hans väg 2  
192 68 Sollentuna, Sweden  
+46 (0)8 505 332 00  
[support@fillauer.com](mailto:support@fillauer.com)

#### Fillauer



2710 Amnicola Highway  
Chattanooga, TN 37406  
423.624.0946  
[www.fillauer.com](http://www.fillauer.com)



#### Fillauer Europe

Kung Hans väg 2  
192 68 Sollentuna, Sweden  
+46 (0)8 505 332 00

